



ResMed

Our softest
mask ever

To find out more visit
ResMed.com/AirTouchF20

AirTouch™ F20



Caring for your AirTouch™ F20 cushion

Featuring ResMed's UltraSoft™ memory foam cushion, the AirTouch F20 has been designed to take comfort to a fresh new level. The cushion, with its new remarkable texture and feeling, requires different maintenance and care compared to traditional silicone, and is recommended to be replaced every month. Please see user guide for full mask and cushion care instructions.



Discard once signs of deterioration form



Replace once signs of deterioration form



Clean with ResMed mask wipes



Keep dry and away from liquids



Do not expose to direct sunlight



Do not use domestic cleaner or domestic cleaning devices



Do not use alcohol wipes



Do not submerge in liquids



ResMed Ltd

1 Elizabeth Macarthur Drive
Bella Vista NSW 2153 Australia

AirTouch and ResMed UltraSoft are trademarks of ResMed Ltd. For patent information, see ResMed.com/ip.
See ResMed.com for other ResMed locations worldwide. Specifications may change without notice. © 2017 ResMed Ltd. 10110373/1 2017-11