




# Lumis 150 ST with Oxygen and invasive circuit set up



## OFF LABEL USE

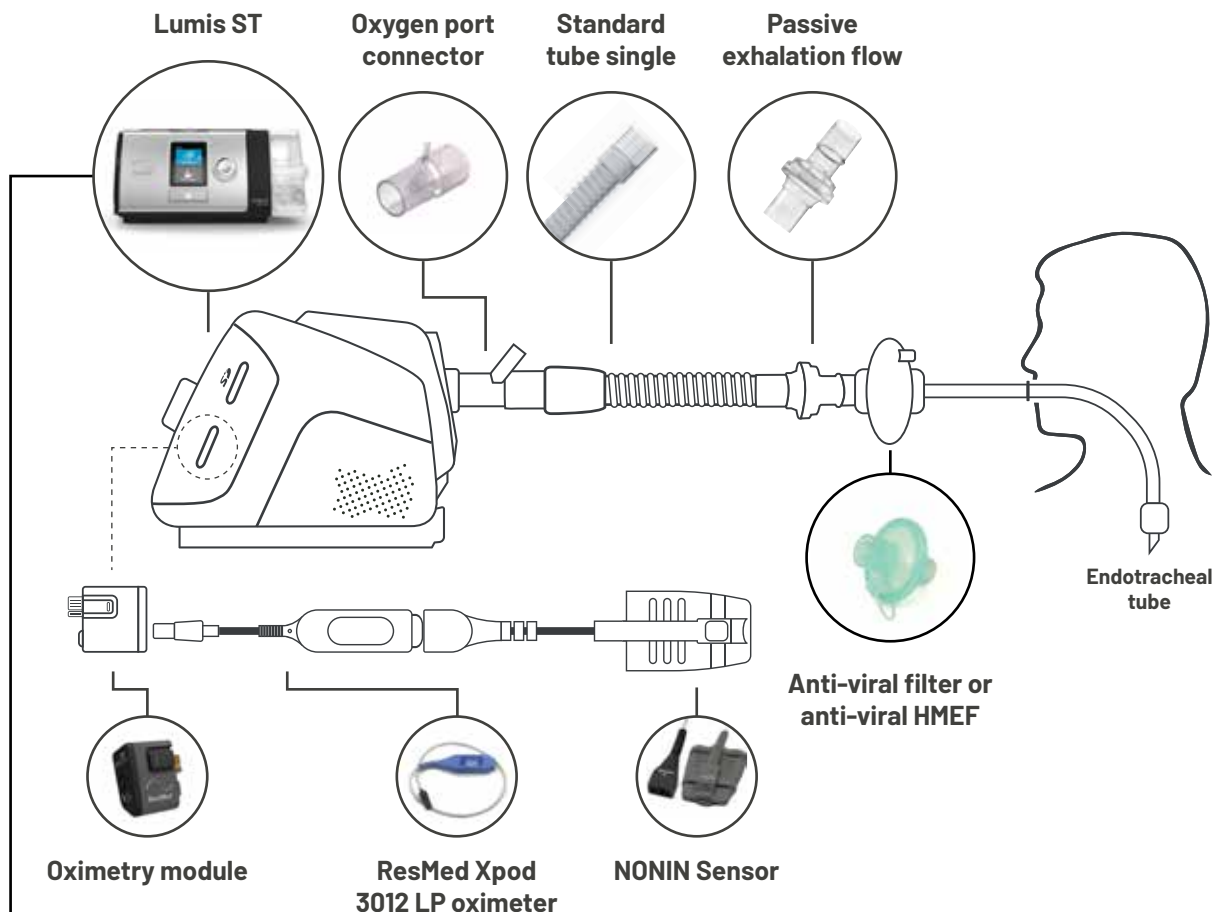

28125 or 28226	<b>Lumis 150 ST or Lumis 150 VPAP ST-A</b>
26960	<b>Oxygen port connector</b>
21948	<b>Standard tube single</b>
24988	<b>Passive exhalation flow</b>
24966 24967	<b>Anti-viral filter (50 pack)</b> <b>Anti-viral HMEF (Heat Moisture Exchange Filter) for adults (50 pack)</b> <b>HMEF for paediatric (50 pack)</b> *Any anti-viral filter can be used with this circuit
24969	
Supplied by hospital	<b>Oxygen tube</b>
Supplied by hospital	<b>Optional connector</b>
Supplied by hospital	<b>Endotracheal tube</b>

37302	<b>Oximetry Module</b>
22374	<b>ResMed Xpod 3012 LP oximeter</b>
22371	<b>Nonin Xpod clip</b>
22338	<b>NONIN 7000A Single Use (Disposable) Oximetry Sensors - 24pk</b>

**Max pressure: 25cm H<sub>2</sub>O**  
**Max backup breath rate: 50bpm**  
**Note: no alarms, only Leak Alert**  
 iVAPS mode is **not** recommended for COVID-19

**Note:** Please ensure the "AB Filter" option is set to "YES" when set up is complete. If using a F&P humidifier, set 'external Humidifier' to 'YES'

### Oxygen note:

- Lumis can take supplemental oxygen up to 15L/min in all modes except iVAPS (4L/min)
- SpO<sub>2</sub> monitoring available on device
- SpO<sub>2</sub> monitoring on device to be used with Oximetry

### Clinical information:

<https://www.resmed.com/au/dam/documents/anz/covid-19/Clinical-Bulletin-Use-of-Bi-level-devices-with-Endotracheal-Tubes.pdf>



ResMed Ltd  
 1 Elizabeth Macarthur Drive  
 Bella Vista NSW 2153 Australia  
 +61(02) 8884 1000 or 1800 658 189